

Instruction for use

Your K-9 Intubation trainer is carefully hand-made and each unit is inspected before shipment. This manikin has arrived in a condition ready for use.

Lubricant

Lubricate with each insertion to avoid damage

Clamp

A clamp included in the scope of delivery offers a firm stand. The clamp can be easily attached to a desk.


Balloon

The stand also offers a success check if the intubation is carried out correctly. If intubation was correct, the balloon will inflate.

Cleaning

You can remove the head from the base.

Grab the head with one hand on the upper part of the snout area. Hold the bottom of base with your other hand. Now, pull apart gently.

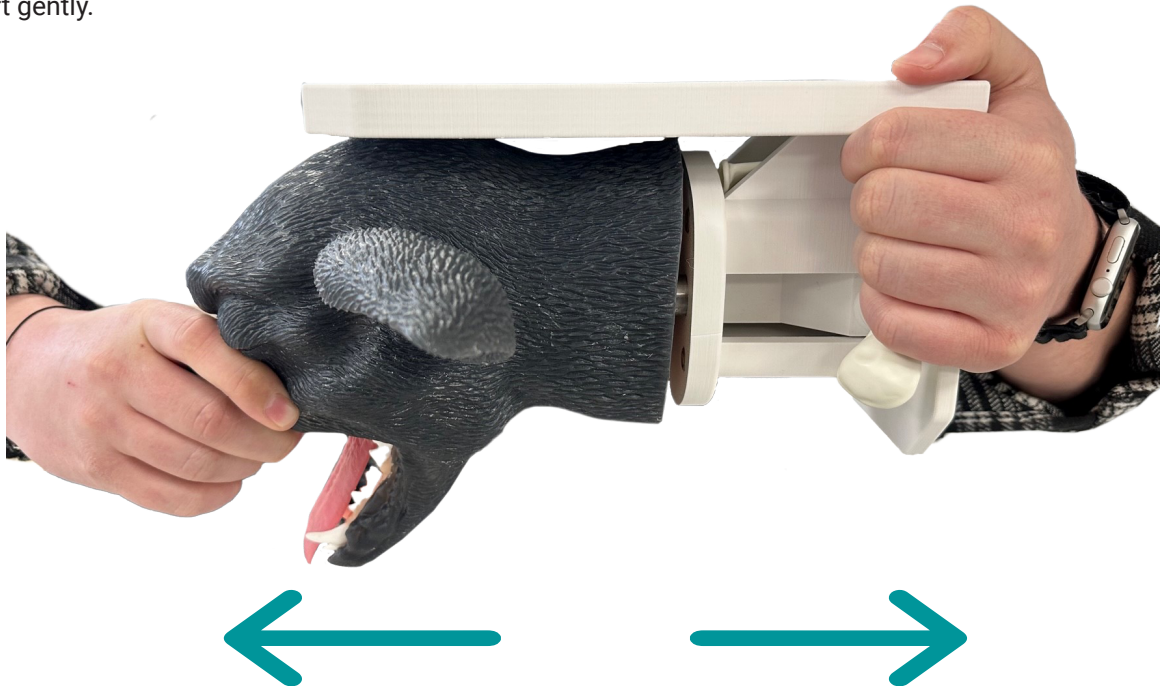

DON'T pull ears or tongue.

You can flush the airway with warm soapy water.

