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Video laryngoscope in combination with single-use blades

Contents

1. Introduction	9
1.1 Description	9
1.2 Equipment	9
1.3 Specifications	9
1.4 Statutory provisions	10
1.5 Environment	10
2. Function	10
2.1 Unpacking and inspecting	10
2.2 Battery	10
2.3 Single-use blade	11
2.4 Using the video laryngoscope	11
2.5 Warming up	12
2.6 Setting the date and time during initial operation	12
2.7 Recording static images	12
2.8 Recording video loops	12
2.9 Reviewing images and videos	12
2.10 Ending video playback	12
3. Cleaning	12
4. Electromagnetic compatibility	13
5. Troubleshooting	13
6. Warranty	13

1. Introduction

1.1 Description

The video laryngoscope for single-use blades REF 91.000.01.002 of the company P. J. Dahlhausen & Co. GmbH is an instrument that serves as an aid during endotracheal intubation. It provides the operator with a clear view of the larynx and trachea on the 3.5 LED screen. The device can be used in the hospital environment or in emergency medicine. Images and videos can be stored on the 4 GB SD card and transferred via cable later on. The video laryngoscope runs on a 3.7 V Lithium battery pack. The device can be used only in combination with the single-use blades.

1.2 Equipment

Single-use blade for video laryngoscope, child size	MAC 1	REF 91.000.03.001
Single-use blade for video laryngoscope, child size	MAC 2	REF 91.000.03.002
Single-use blade for video laryngoscope, adult size	MAC 3	REF 91.000.03.003
Single-use blade for video laryngoscope, adult size	MAC 4	REF 91.000.03.004
Single-use blade for video laryngoscope, adult size difficult airway	MAC DX3	REF 91.000.03.005

Notice

The video laryngoscope may be used only with the single-use blades of the company P. J. Dahlhausen & Co. GmbH. It is not permitted to open up sealed parts of the device, as this adversely impacts the function and endangers patient safety. Any violation shall lead to termination of the warranty.

1.3 Specifications

Video laryngoscope

Display	3.5" LED colour display
Dimensions:	94 mm x 70 mm x 12 mm
Light source:	LED
Monitor:	Liquid Crystal Display (LCD)
Resolution:	640 x 480 (RGB)
Format:	4 : 3
Image display/second:	30 FPS
Camera	Resolution: 2.0 M pixels
	Anti-fog lens
	Illuminance: ≥ 800 LUX
Battery pack	Type: rechargeable 3.7 V lithium battery pack
	Power: 120 minutes
	Capacity: 3,400 mAh
	Charge: approx. 300 times
	Charge time: approx. 8 hours
Connector	Input: 100 – 240 V, 50 – 60 Hz, 0.15 A
	Output: 5 V, 1.0 A
Charge	Micro USB charging cable
Weight	313 g
Memory card	4 GB SD
Materials	ABS (Acrylonitrile Butadiene Styrene)
	Stainless steel
Single-use blade	
Material	PC (Polycarbonate)
Packaging	PE bag

1.4 Statutory provisions

The CE mark is a sign that the requirements laid down in Directive 93/42/EEC and 2007/47/E for medical devices issued by the European Council are met. The product meets the requirements of standards EN 60601-1 and EN 60601-2.

This device may only be used by trained personnel authorised to perform intubations with a video laryngoscope.

The product is classified as a type BF Applied Part in accordance with standard EN 60601-1.

This product must not be disposed of with regular household waste, but has to be brought to a refuse collection point for recycling of electronic devices. Users are encouraged to contact their medical device consultant of the company Dahlhausen & Co. for information about the applicable local regulations governing the disposal of refuse from electronic and electrical installations and similar future environmental regulations.

1.5 Environment

Operating conditions

- +5° - +40°C ambient temperature
- Relative humidity: 10 - 95%

Storage and transport conditions

- -20° - +60°C ambient temperature
- Relative humidity: 10 - 95%

2. Function

2.1 Unpacking and inspecting

Remove the device from the case and check it for visible damages. In case visible damages have occurred during transport, do not use the device and inform your medical device consultant of Dahlhausen u. Co. GmbH. The video laryngoscope is supplied non-sterile. Please disinfect the device prior to the first application (see section 3).

The single-use blade is supplied as a cleanroom-packaged product. Please check whether the precautions for handling the device meet the local, national and harmonised standards. The Dahlhausen single-use blade is supplied and sold separately.

2.2 Battery

The video laryngoscope runs on a 3.7 V Lithium battery pack. The device is switched on or off by pressing the power button. The current battery charge level is displayed in the left lower corner. When the battery pack is fully charged, the battery life is 120 minutes. The battery pack provides the optimum off-grid power for the display and blade and is rechargeable. A reminder is given 5 minutes before the power runs out. The charging time is 8 hours from a fully discharged state. The average useful life is 3 years.

Allowing rechargeable lithium batteries to fully discharge can damage the battery and significantly reduce performance. Total discharge of the battery pack should be avoided.

2.3 Single-use blade

Open the packaging and slide the single-use blade over the camera stick. The blade is in the correct position if there is an audible click and it engages noticeably.

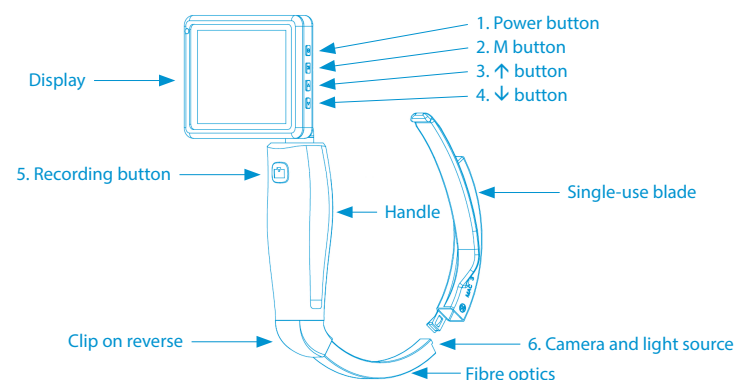
The Dahlhausen single-use blade must only be used in combination with the Dahlhausen video laryngoscope. The blade must be disposed after every treatment.

Do not use the blade in case of visible damages, as this may cause injury of the airways.

2.4 Using the video laryngoscope

The video laryngoscope is designed to be used with conventional laryngoscopy following the usual technique. The user should have completed qualified training in endotracheal intubation techniques. Still images and videos of the intubation procedure can be recorded and automatically saved to the device's SD card.

1. Select the appropriate single-use blade according to the patient's oral cavity size.
2. Assembly: Insert the camera stick into the blade, listen for a „click“ and feel the camera stick engage with the blade.
3. Disassembly: Remove the blade from the camera stick.

1. Power button: Switch the device on/off
2. M button: Menu button / video play and Pause button / Enter / Time selection
3. ↑ button: Last image or video / increase value
4. ↓ button: Next image or video / decrease value
5. Record button: Short press: camera / long press: video / return
6. Camera: Record

● Pressing the „Power button“ (1) for 5 seconds will power on the device and the Dahlhausen logo is displayed for a few seconds. After that, the device is ready for use. If a video blade is correctly fitted to the handle, an image from the camera will appear on the screen.

● Upon switching on the device, for a few seconds the screen will project a small battery symbol icon in the lower left hand corner indicating the power level of the internal rechargeable lithium battery. The date and time will also be displayed.

● If the battery power is low and in need of recharging, the device can be recharged with the external mini USB charging cable. If the battery charge is minimal, the monitor is switched off automatically.

2.5 Warming up

Because the camera lens is heated up by the electrical component of the blade, lens fogging will be cleared immediately after it is inserted into the warm and humid conditions of the airway.

2.6 Setting the date and time during initial operation

- Press the „M“ menu button (2) and select „Time setting“.
- Press the Up or Down arrow buttons to set the date and time.
- Press the „M“ menu button (2) to confirm the changes. The change will be shown in the upper right corner of the display.

2.7 Recording static images

- Briefly press the „Recording“ button (5) on the handle to capture images. If the symbol  appears, images are automatically saved to the SD card in the „JPG“ file format.

Notice:

Static images cannot be captured and saved while the device is recording video.

2.8 Recording video loops

- Press the „Recording“ button (5) for 2 seconds to start recording. The action is noted by an ongoing blinking „Recording“ on the display. The duration of the recording will also be displayed.
- Press the same button a second time to end the recording. The video file will automatically be saved to the SD card under an „MP4“ format.

2.9 Reviewing images and videos

- Both the static images and video loops can be reviewed by pressing the „M“ menu button (2) for selecting „Image view“ or „Video play“ to preview.
- Each file is marked with a date and time stamp.

2.10 Ending video playback

- Press the „M“ menu button (2) to pause the video playback. End the video playback by pressing the „Recording“ button (5).

Notice:

Owing to its emissions characteristics, the video laryngoscope is suitable for use in the industrial environment and in hospitals (CISPR 11, Class A). When used in a residential environment (for which CISPR 11 Class B is normally required), the device might not be adequately protected from radiofrequency communication services. The user might need to take mitigation measures, such as relocating or re-orienting the device.

3. Cleaning

The device must be disinfected before it is used for the first time.

3.1 Cleaning the handle and video laryngoscope device display

The screen is not waterproof. For this reason, it must never be submerged in or exposed to excessive amounts of fluids. The face of the device, including the front control buttons, is sealed and can be disinfected with appropriate antimicrobial disinfectant wipes, such as Descosept Sensitive Wipes of Dr. Schumacher. The remaining surfaces of the device are cleaned in the same way as the screen. After every application, the system is to be disinfected according to the local guidelines.

3.2 Single-use blade

The blade is a disposable product that has to be disposed of in accordance with the hospital's hygiene regulations. Re-use leads to multiplication and spreading of germs. Sterilisation and cleaning (for re-use) can impair the function and mode of operation of the product and cause a hazard for the patient.

4. Electromagnetic compatibility

The video laryngoscope meets the requirements laid down in standard IEC 60601-1-2 regarding electromagnetic compatibility; during application, the user must observe the electromagnetic compatibility requirements.

Because portable and mobile RF communication equipment may affect the performance of the video laryngoscope, it should not be used near mobile phones, microwave ovens or similar devices in order to avoid electromagnetic interference.

The video laryngoscope should not be operated when stacked with or in the direct vicinity of other devices. If this cannot be avoided, the device should be monitored in order to verify normal operation in the configuration.

When a device class intended for use in industrial environments because of its conduction and radiation interference is used in other environments, the electromagnetic compatibility has to be ensured.

Using non-original accessories and non-original cables can lead to higher emissions or a reduced interference resistance of the video laryngoscope.

5. Troubleshooting

The following advice is offered to help deal with unexpected device conditions:

5.1 No image shown on the screen

- Check the power supply.
- Restart if necessary.

5.2 The image appears on the screen, but is grainy or dim

- Make sure that the handle battery is adequately charged.
- Check all blade and handle connections.
- Ensure that lenses on camera and LEDs on blade are clean.

6. Warranty

When used under normal operating conditions, we offer a guarantee on defects in materials or production for 12 months from the date of purchase:

What is covered: replacement of all parts.

What is not covered: transportation charges as well as damage caused by incorrect use or misuse, accident, negligence and battery pack.

For warranty claims: Please contact our medical device consultants, who will then review your claim. If applicable, we will issue an approval for returning the products. When returning the video laryngoscope, you must provide proof of disinfection.

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