Rollbord™

**User's Guide
Gebrauchsanweisung
Mode d'emploi
Manuale d'uso
Instrucciones de uso
Gebruiksaanwijzing
Bruksanvisning**

Warranty

Do not dispose!

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Ne jetez pas!

No deseche los!

Non smaltire!

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Släng inte!

!ATTENTION / IMPORTANT !

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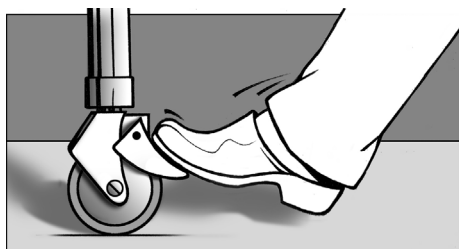
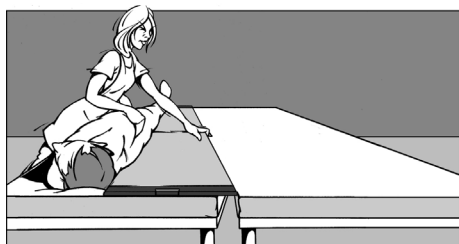
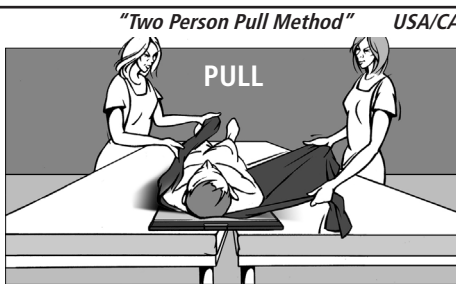
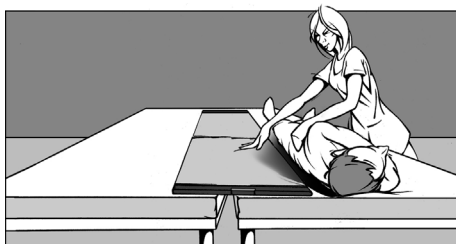
7. Turn patient and remove „Rollbord“

Instructions

The Samarit „Rollbord“ is highly efficient, requiring only 15% of the patient's body weight to either push or pull the patient from one lateral surface to the other. One staff member can easily move an average sized patient with minimal effort.

CAUTION: Do not become over-confident and attempt to move patients above your capacity. With large or touch-sensitive patients, request assistance and use the *Two Person Pull* method. Always use your proper body position for any patient transfer.

Picture 1. The „Rollbord“ weighs only 6 lbs. (3 kg) and can be carried on the shoulder with the convenient adjustable strap leaving both hands free for stretcher operation.

Picture 2. For easier transfer, ensure that surfaces are leveled or that the receiving surface is slightly lower. Place the „Rollbord“ and position next to patient.

Picture 3. Always ensure that brakes are operational and are locked on both vehicles.

Picture 4. Roll the patient towards you and move the „Rollbord“ to his back. For tall patients, either place a pillow under shoulder/head and under patient's lower legs to elevate heels.

Picture 5. Ease patient back onto „Rollbord“ so that about 1/3 of patient is on the „Rollbord“. For wider surfaces, place less of the patient onto the „Rollbord“; narrower surfaces more.

„One Person Push Method“

Picture 6.1 Place hands under patient's hip and shoulder. Pushing there positions the centre of patient's weight onto the „Rollbord“.

„Two Person Pull Method“

USA/CA

Picture 6.2 Place pull sheet using well-known method under the patient and raise both ends. The patient is then carefully pulled by two persons over the „Rollbord“ (preferable method for heavy patients).

Picture 7. Once centered, roll patient towards you and slide „Rollbord“ from under the patient with one hand. Pull away side under patient's leg first. Remove „Rollbord“ and disinfect, if necessary.

Caution: Do not pull „Rollbord“ by the handles!

CLEANING: The „Rollbord“ can be cleaned best with *„Incidin® OxyWipe“* by ECOLAB or any hospital disinfectant (**no chlorine, no concentrate, no bleach**). For anticipated soiled conditions use the „Rollbord“ **disposable cover**. It covers ends of „Rollbord“ and allows any residue to be folded untouched into the cover, leaving a clean surface. For replacement of the silver cover, simply unscrew one cap-end and exchange cover. Repairs should only be carried out by authorised personnel.

For more information visit our website and watch the instruction videos

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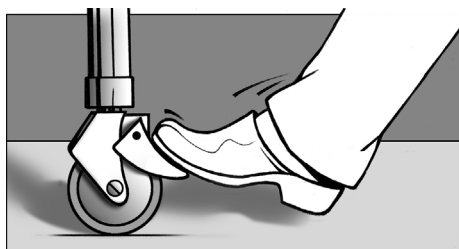
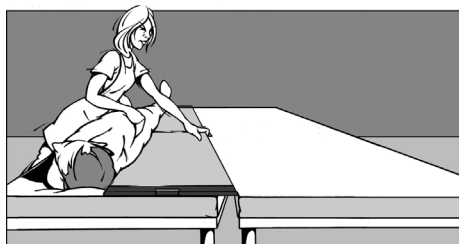
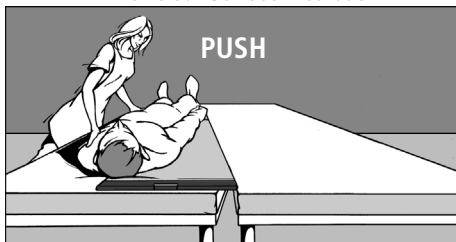
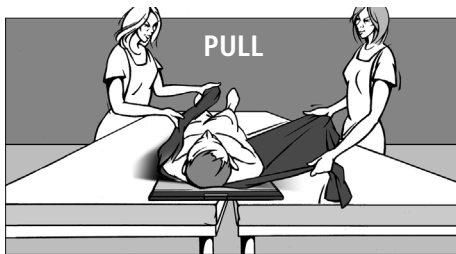
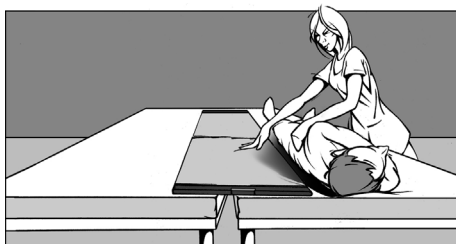
2 Models

WEIGHT LIMIT

„Hospitals ask us to give a weight limit. We like to explain that this is a theoretic number because such limit depends on support, surface of support, condition and volume of patient, use of pull sheet, age and condition of the product and finally the experience of the user.”

OEM Products upon request, www.samarit.com

Consult our website
www.samarit.com
and find out more about our interesting product lines!

LATERAL TRANSFER

SEATED TRANSFER

NURSING AIDS

AMBULANCE

BEACH-CHAIR DEVICE

**BEACH-CHAIR NON-
SLIP CUSHION**

**PATIENT TURNING
SYSTEM «PTS»**

**PATIENT FACE
PROTECTOR**

Safety information

The user is responsible to read the „User's Guide“ accompanying every product delivered (original box). Incorrect use is not covered by our liability. For instruction consult our Video Clip at www.samarit.com and / or our distributor.

The user has to make sure that the product is in perfect working condition to meet the Hospital safety regulations. A transfer aid (Rollbord) failing to meet the required demand must be removed from service immediately.

For repair call our Service Number or contact your nearest distributor.

© 2017 All rights reserved by Samarit Medical AG, Switzerland. Products are subject to change without prior notice. Samarit Medical AG, Switzerland provides the information in this booklet to be read by anyone, but retains the copyright on all text and graphics. „Rollbord™“ is a registered trademark of Samarit Medical AG, Switzerland.

Limited Warranty

by SAMARIT MEDICAL AG SWITZERLAND & CANADA

Customer: _____

Date of Purchase: _____

Phone: _____

E-Mail: _____

Order Number: _____

Product: _____

Serial Number: _____

WARRANTY

SAMARIT MEDICAL AG warrants your product to be free from defects in material and workmanship for a period of 12 / 24 months from the original date of purchase. If you discover a defect in a product covered by this warranty, we will repair at our option using new or refurbished components, or if repair is not possible, replace the item.

EXCLUSIONS

This warranty covers defects in manufacturing discovered while using the product as recommended by the manufacturer. The warranty does not cover loss or theft, nor does coverage extend to damage caused by misuse, abuse, unauthorized modification, improper storage conditions, lightening, or natural disasters. The warranty does not cover parts that are subject to normal wear and tear replacement requirements, such as.

LIMITS OF LIABILITY

Should the product(s) fail, your sole recourse shall be repair or replacement, as described in the preceding paragraphs. We will not be held liable to you or any other party for any damages that result from the failure of this product. Damages excluded include, but are not limited to, the following: lost profits, lost savings, lost data, damage to other equipment, and incidental or consequential damages arising from the use, or inability to use this product. In no event will SAMARIT MEDICAL AG be liable for more than the amount of your purchase price, not to exceed the current list price of the product, and excluding tax, shipping and handling charges.

SAMARIT MEDICAL AG disclaims any other warranties, express or implied.
By installing or using the product, the user accepts all terms described herein.

HOW TO OBTAIN SERVICE UNDER THIS WARRANTY

Contact us priority: info@samarit.com

- ROLLBORD HIGHTEC	36 months warranty
- ROLLBORD PROFESSIONAL, BLUE TRANSFER BOARD, ROLLBORD ECOLITE	24 months warranty
- H-LINE, NURSING AIDS	12 months warranty

REQUIREMENTS

Copy of invoice with serial no., pictures of product damage send to info@samarit.com.
The cost of shipping to the manufacturer, or authorized repair center or payment of any customs clearance fees or duties are the responsibility of the user. Return shipping costs to be paid for by SAMARIT MEDICAL AG, only with written acceptance from Samarit.

Samarit Medical AG
Gewerbestrasse 12
CH-8132 Egg, Switzerland

Tel.: +41 44 918 10 11 - Fax: +41 44 918 24 39 - info@samarit.com - www.samarit.com