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Content

Page

1.Important information	8
1.1 Intended use of the product.	8
1.2 User groups.	8
1.3 Warranty terms	8
1.4 Declaration of conformity	8
2. Information on the safety and protection of the product	9
3. Use of the ultraHEAD-LOCK II	9
4. Preparation for reuse	10
5. Cleaning and Care	11
6. Maintenance	11
7. Storage	12
8. Service life	12
9. Optional accessories	12
10. Identification and Warranty Certificate	13

1. Important information

Before using the ultraHEAD-LOCK II, please read the instructions for use and safety instructions carefully. Keep the instructions for use in a safe place to avoid operating errors.

Should you require further information or encounter difficulties in its use or maintenance, please contact your local dealership or ultraMEDIC GmbH directly. Always specify the LOT- or serial number to identify the ultraHEAD-LOCK II.

1.1 Intended use of the product

The ultraHEAD-LOCK II is a medical device.

Indication: The ultraHEAD-LOCK II is a head restraint system for use with spinal boards or scoop stretchers, it serves to immobilize the cervical spine (C-spine) in cases of suspected spinal injury.

WARNING! The stabilization of vital functions (airway and circulation) always takes priority over immobilization.

Absolute Contraindications (do not use):

- Open or unstable fractures of the facial skull or skull base;
- In cases of difficult mask ventilation or intubation – if airway management is impeded by head immobilization.

Relative Contraindications (use with caution):

- Severe agitation or panic;
- Severe facial or cranial injuries where immobilization may obstruct the airway, wound care, or visual assessment of critical injuries;
- Vomiting or high risk of aspiration;
- Massive swelling of the face or neck (risk of airway compression or poor mask fit during resuscitation).

1.2 User groups

This product may only be used by qualified, expert personnel. A prerequisite is theoretical and practical experience in handling head restraint systems.

1.3 Warranty terms

The manufacturer grants a warranty period of 24 months on this product. The warranty only encompasses the costs of the spare parts.

The following points are not covered by the warranty: a) possibly omitted shipping or labour costs incurred on the purchaser's side; b) defects in the product due to normal wear and tear; c) damages caused by the user; d) defects or damages caused by non-compliance with the instructions in this document; e) any modifications or repairs to the product without prior approval by ultraMEDIC GmbH.

1.4. Declaration of conformity

The ultraHEAD-LOCK II was manufactured in accordance with the current EU regulation 2017/745. A declaration of conformity for these products can be acquired from the local dealership or the manufacturer.

2. Information on the safety and protection of the product

1. Before and after every use, check the components and patient restraining belts for damage and completeness.
2. Ensure that the patient restraining belts are correctly installed.
3. Before initial use, conduct a test run with the ultraHEAD-LOCK II to become aware of the handling of the product.
4. In the event of damage or suspected damage to the straps or the ultraHEAD-LOCK II, remove the product from operation immediately until any occurring irregularities have been remedied.
5. All serious incidents related to the device shall be reported to the manufacturer and to the competent authority of the Member State where the user and/or the patient is established.

3. Use of the ultraHEAD-LOCK II

Manual immobilisation may not be interrupted when applying the ultraHEAD-LOCK II on the patient.

1. Attach the base plate to the spinal board with the help of the broad restraint strap (Figure 1a,b).

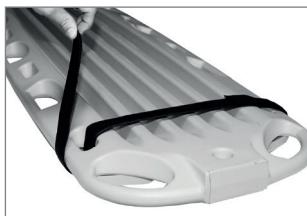
Figure 1a

Fixing the base plate with the wide fastening strap on the back of the spineboard

Figure 1b

Fixed base plate on the front of the spineboard

2. Place the patient onto the spinal board so that the head rests in the centre of the base plate. To ensure a neutral alignment, additional padding may be required under the head (in adults) or the thorax (in children).
3. Open both restraint frame handles and guide the restraint frames into the rails up to the patient's head, then align the recesses with the ears.

4. To arrest, push both handles downwards. (Figure 2b). For possible adjustments, pull the handles upwards and adjust the restraint frames (Figure 2a).

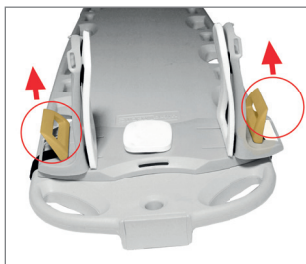
Figure 2a

Handles of the fixing frame downwards - restraint frame movable

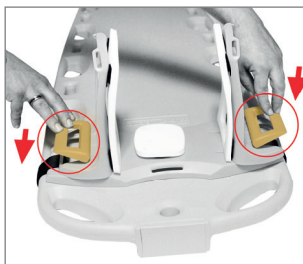
Figure 2b

Handles of the restraint frame upwards - restraint frame locked.

5. Attach both narrow restraint straps to the restraint frame. Centre the straps above the chin and forehead of the patient (Figure 3).

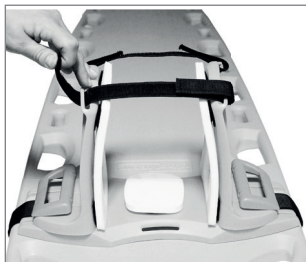
Figure 3

Attachment of the narrow fixing straps

6. Before use, ensure that all straps are fastened tightly.

4. Preparation for reuse

• Dismantling the equipment:

Remove both restraint frames from the base plate, and the base plate from the rescue board.

Parts can be reused after they have been appropriately cleaned. If pads and narrow fixing straps are very heavily contaminated with body fluids, replace them.

• Brushing or immersing in disinfectant:

Brush all parts with a warm water and dish-washing liquid solution for 4 minutes or

immerse them in disinfectant (at 38°C with a concentration of 0.5%).

- **Rinsing and drying:**

Rinse all parts with clean, warm (38°C) water and leave everything to air-dry completely at room temperature.

- **Attaching the padding:**

If required, attach spare padding and straps.

- **Reattaching the base plate:**

Reattach the base plate to the rescue board and store the restraint frames.

5. Cleaning and Care

Never use products with aggressive agents (bleach, ammonia, etc.) for cleaning, avoid the use of scratching materials to remove stains such as brushes, steel wool and knife blades.

For **disinfection**, you can use commercially available alcohol-based agents. Other solutions or cleaning chemicals may damage the material.

Straps: The straps that have gotten wet should only dry naturally. Avoid direct heat radiation from heat sources. These factors could damage the material.

Plastic parts: Clean with a sponge drenched with water diluted with disinfectant and then rinse and dry completely.

6. Maintenance

In order to ensure the safety of the user and the injured party during a rescue mission, we recommend that you have the ultraHEAD-LOCK II and straps visually inspected by an expert on an **annual** basis.

For German users, we recommend consideration to the following regulations and guidelines of MPBetreibV.

For users outside of Germany, a regular check according to country-specific requirements is required.

The following parts must be checked and the following defects must be excluded:

Base plate	deformation, tears, breaks, broken parts
Straps	Cutting and abrasion points, damaged seams, tissue-damaging stains
Plastic parts	Deformation, tears, breaks
other damages	

In the event of damage or suspected damages, remove the product from operation immediately until any occurring irregularities have been remedied.

7. Storage

Store the ultraHEAD-LOCK II in a dry and dark place. Avoid direct sunlight and direct heat radiation from heat sources. These factors can damage the material.

Protect the blocking devices and moving parts from dust and dirt deposits to maintain their maximum effectiveness during use.

8. Service life

The service life of the ultraHEAD-LOCK II depends on the individual conditions of use. Depending on how frequently and intensely the ultraHEAD-LOCK II is used, wear and tear on the product is normal. In this regard, correct use is assumed. The material is subject to an ageing process even if handled carefully. The product should be discarded depending on its condition.

Service life of the straps is set at maximum of **10 years** from commissioning. For the service life of the optional accessories, please refer to the relevant instructions for use.

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